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Subject: Requested analysis of Intuitive device(s) returned via RMA 1430871

Dear Customer,

Intuitive would like to thank you for taking the time to return the device(s) for evaluation and for bringing this concern to our attention.

We have completed our investigation of the returned device(s) based on the reported issue(s) and have provided a summary of the requested analysis for you in this letter.

Analysis Summary Results:

Intuitive Complaint	:	700634386
RMA Number	:	1430871
Date of Event	:	April 29, 2021
Customer Symptom	:	Vision Problem
Symptom Detail	:	Error Message/Fault
Product	:	8MM ENDOSCOPE, 0 DEGREES
Product Part Number	:	470026-64
Serial Number	:	SF2002216
Billing Decision	:	Full Bill
Customer Reported Event	:	ERROR 45312

Failure Analysis Results:

Intuitive has received the product associated with this complaint and completed investigations. During failure analysis investigations, the reported complaint of "ERROR 45312" was confirmed, but not replicated. There was a good image in both the eyes. Cracks/holes were found in the material used to seal off the paddleboard (PCA) at the integrated connector. The endoscope was found with loose desiccant pellets inside of the housing. The optical lens(es) were no longer attached to the optics.

Failure Analysis Image(s):

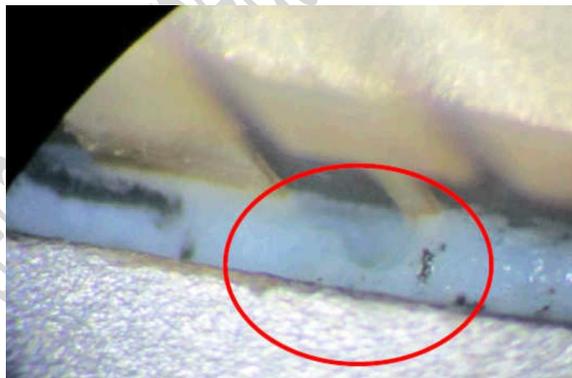
Screen Display – No Trouble Found

Good Image – Left Eye

Good Image – Right Eye

Cracks in the Seal

Loose Desiccant Pellets

Detached Optics

Potential Cause(s) of Cracks in the Seal:

- This failure mode is most likely related to cleaning during reprocessing resulting in damage at the integrated connector. Cracks or holes in the silicone material may allow moisture into the device. If moisture is present on the circuit board during plug-in, shorts can develop and could cause irreversible damage to the device.

Recommendation(s):

- Refer to the Reprocessing Instruments User Manual for detailed reprocessing instructions.
- Use only the validated sterilization steps in the User Manual.
- Do not use metal brushes or abrasive materials for cleaning. This can lead to a damaged surface or coating. Nylon brushes are recommended.
- Do not autoclave Intuitive Surgical endoscopes. Autoclave cycles introduce high temperatures and sudden temperature changes, which will damage the endoscope.
- When reprocessing Intuitive Surgical endoscopes, temperatures should not exceed 140 °F (60 °C).

Potential Cause of Loose Desiccant Pellets:

- The endoscope was manufactured with an older version of the desiccant container, which is more susceptible for loose desiccant pellets to fall out.

Potential Cause(s) of Damaged Optics:

- Mechanical Impact/Shock-Accidental drop of the scope or inadvertent collisions with hard surfaces, resulting in damaged optical components.

Recommendation(s):

- The endoscope is a sensitive device that may become damaged if transported improperly. Ensure the endoscope is transported in the sterilization tray or another means to protect the tip from impacting hard surfaces.
- Endoscopes should be handled with care during the cleaning process, paying special attention to inadvertent collisions.
- Endoscopes should be inspected before and after cleaning for any physical damages to outer surface and to ensure the distal windows are intact and firmly in place.

We would like to take this opportunity to reiterate the importance of closely following the Instructions for Use (IFU) guidelines to ensure adequate handling of our products. Furthermore, this will enable the preservation of the expected performance of our products throughout their lives.

We appreciate your input as it helps Intuitive in continuously improving our products and services to you. If you have any questions, or would like to discuss your concern further, please do not hesitate to contact me at the email address below or contact our Customer Service department.

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